XCP-DS[®]

Sensor Positioning System for EzSensor

VATECH Co., Ltd
473-4, Bora-dong, Giheung-gu, Yongin-si,
Gyeonggi-do, Korea 446-904
Tel : 82 31 285 6950
Fax : 82 31 286 3007
www.vatech.co.kr

E-WOO Technology UK Ltd.
Axiom House, The Centre Feltham, Middlesex,
TW13 4AU, United Kingdom
Tel : 44 20 8831 1660
Fax : 44 20 8831 1679

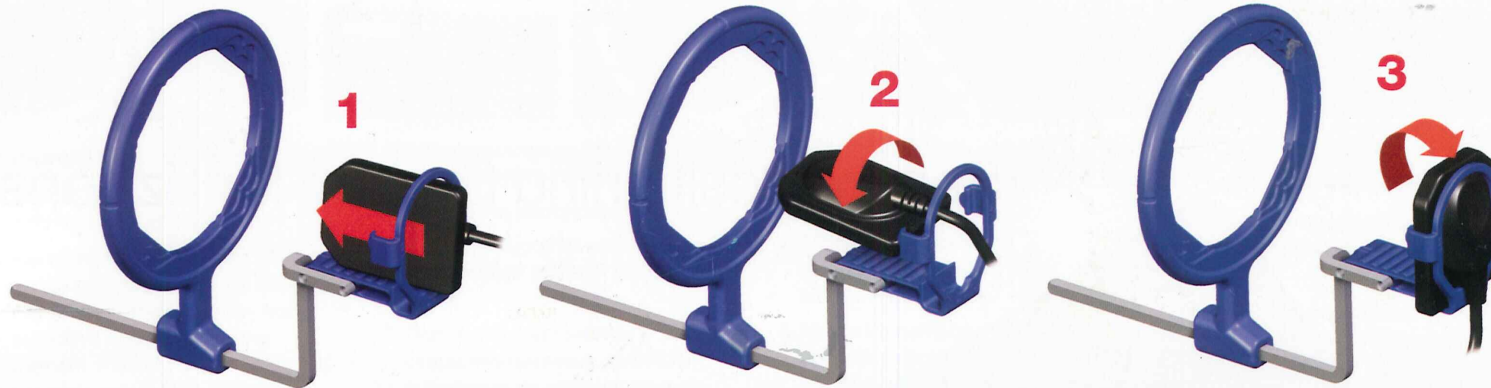
DENTSPLY
RINN

1212 Abbott Drive
Elgin, Illinois 60123-1819
1 847 742 1115
1 847 742 1122 Fax
www.rinncorp.com

552800 XCP-DS for EzSensor, Size 1.5
055913 XCP-DS Instruction Poster for EzSensor

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



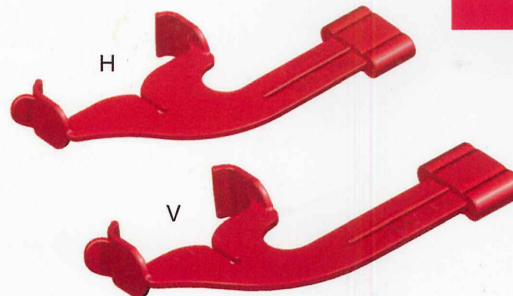
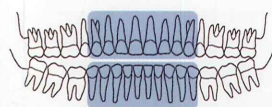
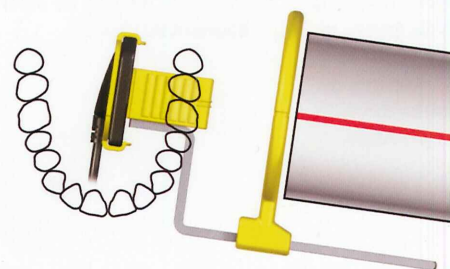
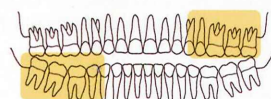
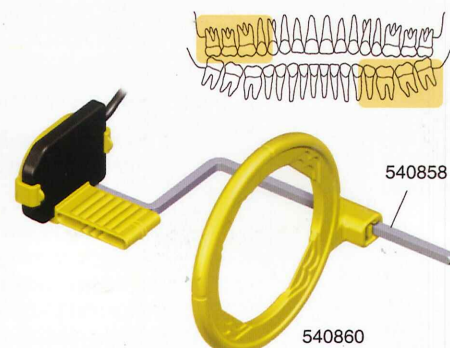
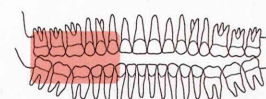
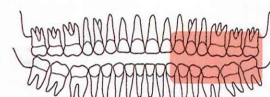
Bitewing
Моляры
Bitewing
Bitewing
Bitewing

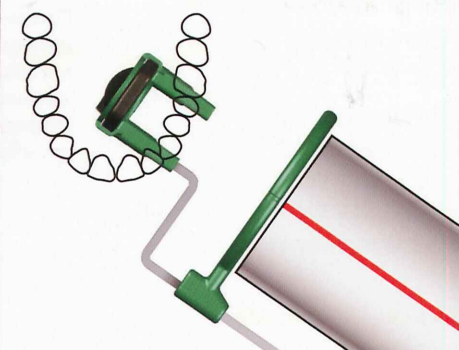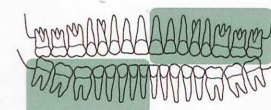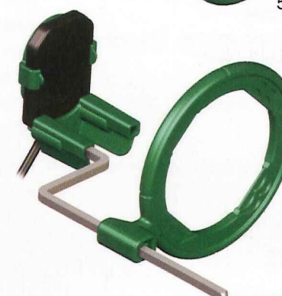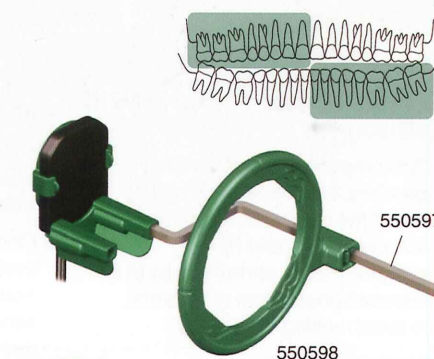Endodontic
Эндодонтическое
Endodontie
Endodoncia
Endodonzia

Cleaning and Sterilization Procedure

Prior to use, always cover the sensor with a disposable hygienic protective cover before inserting the sensor into the holder. Discard cover after use. After use with a patient, the parts of the positioning system must be cleaned and sterilized. **DO NOT AUTOCLAVE SENSORS.**

1. Disassemble instrument(s).
2. Remove debris from components with hot water and soap.
3. Put components in sterilization pouches, and place in the middle tray of the autoclave, away from autoclave walls and heating element. Plastic parts must be in a separate pouch from metal arms, to avoid melting or warping.
4. Cycle per autoclave manufacturer's instructions. Do not exceed 270° F (132° C).
 - DO NOT use phenol-based glutaraldehyde.
 - DO NOT use special application or ultrasonic cleaners.
 - DO NOT CHEMIClave or DRY-HEAT STERILIZE.
 - DO NOT cold sterilize.

Note: Plastic parts have a limited life and should be replaced periodically. Any method of sterilization will shorten the life of plastic parts.

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